

### ***Product Description***

Product Code: SM-RE201

P2/N95 E-MED - Healthcare Particulate Surgical Respirator with Earloops

Fluid Resistance – Level 3

P2/N95 E-Med Particulate Surgical Respirator is flat folded respirator with low profile design, which contours with face and accommodate facial movements without compromising the fit.

5 PLY, P2/N95 E-Med Particulate Surgical Respirator is designed with two layers of melt blown layers that act as a filter to contain potentially harmful airborne particles and contaminants, while outer spun-bond layer is fluid resistant and inner layer absorb moisture and provides comfort to the wearer in hot and humid conditions. An additional outer spun bond layer helps the respirator to hold its shape without deformation when used.

P2/N95 E-Med Particulate Surgical Respirator is supplied in a non sterile state.

### ***Intended Use***

P2/N95 E-Med Particulate Surgical Respirator is intended for single use only by general public or in healthcare setting to protect the wearer from airborne particles and from liquid contaminating the face. . It is intended to cover the nose and mouth of the wearer to help reduce wearer exposure to pathogenic biological airborne particulates and also helps to protect the wearer from splash and spray of body fluids.



***Design Principles***

P2/N95 E-Med Particulate Surgical Respirator is a made from the following materials:

- Spunbond 65gsm
- Spunbond 50gsm
- Meltblown 25gsm
- Meltblown 25gsm
- Spunbond 30gsm

***Packaging specifications***

- Items per box: 20
- Boxes per carton: 28
- Items per carton: 560
- Cartons per pallet: 18
- Items per pallet: 10,080

## **Instructions for Use**

### **Donning**

- Wash your hands thoroughly before and after putting on a respirator.
- To work as intended, a mask needs to fit your face properly. To ensure the correct fit:
  - A. Separate the edges of the mask to fully open it
  - B. Bend the nose clip to form a gentle curve. The nose clip represents the top of the mask
  - C. Hold the mask upside down to expose the two Ear Loops
  - D. Using your index fingers and thumbs, separate the two Ear Loops
  - E. While holding the Ear Loops cup the mask under your chin
  - F. Hold the mask by both ear loops and place one loop over each ear
  - G. Gently conform/press the nosepiece across the bridge of your nose by pressing down with fingers until the fit is snug
  - H. Continue to adjust the mask until you feel you have achieved a good and comfortable facial fit.

### **Doffing**

- Hold both of the ear loops and gently lift and pull the mask away from you and away from your face.
- Discard immediately according to local policies and procedures. Perform hand hygiene.